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en
Description: The Kangaroo™ Gastrostomy Feeding Tube with Y-Port is made of medical-grade silicone. Now with new and improved clear silicone with tapered flush tip design. A 2.7 g packet of water-soluble jelly is also included. The G-tube's graduated shaft is equipped with a gastric balloon for internal retention.

Accuracy of graduations is ± 0.41 cm.

Note: For enteral feeding only

Indications: This tube is indicated for use as a replacement tube for the administration of nutrition, fluids, and medications to the stomach in patients physically unable to manage nutritional intake through normal mastication and deglutition.

Contraindications: The use of this product is contraindicated in patients with known sensitivities or allergies to its components.

Warnings: 1. Do not use if package is opened or damaged.
2. The comfort and convenience of gastric feeding must be weighed against the risk of regurgitation or aspiration of stomach contents. The use of enteral feeding should be considered for any patient who in the absence of a trained health care professional is at risk of regurgitation or aspiration.

3. This device should not be inserted by a trained clinician.
4. Do not use for balloon inflation.
5. Do not exceed recommended balloon inflation volumes.
6. This is a disposable device for single use. Do not reuse.

Precautions: 1. Do not autoclave.

2. To prevent clogging, feeding tubes should be flushed frequently using a 35 mL (or larger) syringe at the following times:
a. before and after each feeding
b. before and after administering medication
c. once every four hours during continuous feeding or between intermittent feedings
d. each time the tube is disconnected
e. each time the tube is filled/changed
f. each time the pump is stopped.

3. Use only tap or sterile water to flush. **DO NOT** use juice, colas, or solutions containing meat tenderizer to flush or open a clogged feeding tube.

4. Never use a needle or sharp object to unclog a tube.

5. Replace tube at four (4) week intervals to ensure optimum tube patency.

6. Use a Kangaroo enteral feeding pump for accuracy and control of nutritional formula delivery. Infusion pumps that deliver in excess of 40psi should not be used as excessive pressure is capable of rupturing and pump sets to balloon and/or rupture. Consult pump manufacturer's specifications and recommendations.

7. Administration of medications should always be guided by hospital policy. Many liquid preparations contain Sorbitol which tends to interact with enteral formulas and clog the feeding tube. Thoroughly wash tablets, excluding enteric tablets which should never be crushed; however, always consult with your physician which tablets should be avoided for feeding through a gastrostomy.

8. Assess balloon inflation daily by withdrawing 3-5 mL of water from the inflation lumen. If water contains gastric contents, the balloon has been damaged and the catheter must be replaced. If it does not contain gastric contents, reinject the water.

9. Ensure stomach tract is intact prior to inserting this tube.

Directions: Read all warnings and precautions prior to tube insertion.

1. Insert the tube into the stoma.

2. Lubricate G-tube with enclosed lubricating oil. Use petroleum based jelly on the G-Tube.

3. Gently insert tip of G-tube through stoma tract, advancing until the distal 6 cm (2.5") of the tube are fully intraluminal.

4. Balloon inflation - Do not use for inflation.

A. To inflate a 20 mL balloon, inject 20 mL of water or saline into the gastrostomy tube. Do not exceed 20 mL.

B. To inflate a 5 mL balloon, inject 5 mL of water or saline into the gastrostomy tube. Do not exceed 5 mL.

5. When the balloon is inflated, the balloon shaft will remain against the gastric mucosa.

6. Slide the extender disc counter-clockwise against the abdominal wall allowing for in and out play and daily rotation of tube.

7. Confirm intragastric placement prior to initiating feeding.

8. To irrigate the tube, place feeding tube, hold and close roller clamp before opening Y-port.

9. To irrigate using catheter tip, spray open Y-port and insert tip of syringe.

10. After use, remove syringe and recap Y-port. Open roller clamp and restart feeding.

11. To remove the gastrostomy tube, deflate the balloon by injecting air into the luer tip of the syringe into the inflation valve and aspirate balloon contents. If deflation difficulties are encountered, the inflation valve arm should be severed. Gentle traction from the abdomen should be sufficient for catheter removal.

How Supplied: The Kangaroo Gastrostomy Feeding Tube with Y-Port is supplied as follows:

Reorder No.	French Size	Balloon Size	No. in Case
8884-			
715122	4.0 mm (12 Fr/Ch)	5 mL	5
715148	4.7 mm (14 Fr/Ch)	5 mL	5
720163	5.3 mm (16 Fr/Ch)	20 mL	5
720189	6.0 mm (18 Fr/Ch)	20 mL	5
720205	6.7 mm (20 Fr/Ch)	20 mL	5
720221	7.3 mm (22 Fr/Ch)	20 mL	5
720247	8.0 mm (24 Fr/Ch)	20 mL	5
720265	8.7 mm (26 Fr/Ch)	20 mL	5
720285	9.3 mm (28 Fr/Ch)	20 mL	5

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